

SHIPPING INSERT

PLASMA TREATMENT WET SHIPPING IN BAUSCH & LOMB BOSTON SIMPLUS® MULTI-ACTION SOLUTION FOR UP TO 30 DAYS

Description:

Plasma treated Bausch & Lomb Boston® GP contact lens(es) are shipped in Boston SIMPLUS Multi-Action Solution, a sterile, aqueous, buffered solution that contains poloxamine, hydroxyalkyl phosphonate, boric acid, sodium borate, sodium chloride, hydroxypropylmethyl cellulose, Glucam and preserved with polyaminopropyl biguanide (0.0005%) and chlorhexidine gluconate (0.003%).

CAUTION: *Non-sterile. Clean and condition lenses prior to use. USE ONLY BOSTON SIMPLUS MULTI-ACTION SOLUTION TO CLEAN AND CONDITION THE LENSES.*

Precautions for Eye Care Practitioners:

- The plasma treated Boston GP lenses are packaged non-sterile in a preserved aqueous solution, Boston SIMPLUS Multi-Action Solution. The solution acts as a wetting, soaking, conditioning and disinfecting agent. If the patient has experienced a prior history of allergy to any of the ingredients in Boston SIMPLUS Multi-Action Solution, remove the lens from the solution and soak the lens 24 hours in unpreserved saline prior to cleaning, disinfecting and dispensing.
- Prior to dispensing, clean and disinfect the lens(es) according to the appropriate lens care regimen. Only Boston SIMPLUS Multi-Action Solution should be used for cleaning the plasma treated Boston® lenses.
- Never reuse the solution. You may store the lenses in the unopened container until ready to dispense, up to a maximum of thirty days from the date of filling (see lens shipping carton label). If the lenses are stored for longer periods of time, they should be cleaned and disinfected with Boston SIMPLUS Multi-Action Solution.
- Inspect package for leakage or any other damage from shipping.
- Return any damaged packages to your Authorized Boston Manufacturer.

Note: *For additional information, please refer to the Bausch & Lomb Boston GP Package Insert.*

Patient Lens Care:

- Eyecare practitioners should review with the patient all lens care directions, including both basic lens care information and specific instructions on the lens care regimen recommended for the patient.
 - Always use fresh rinsing, disinfecting and storage solutions.
 - Always wash your hands before handling your lenses.
 - Always keep the lenses completely immersed in the recommended storage solution when the lenses are not being worn.
 - Follow the directions from the package inserts of the lens care solutions.
- Failure to adhere to these procedures may result in the development of serious ocular complications.